

Bioway Chemistry Calibrator Series

Hemoglobin A1c Calibrator Kit

4*0.5ml Dry Powder



Cat. No. R032S00

HbA1c calibrator Kit

INTENDED USE

Bioway Chemistry Calibrator Series Hemoglobin A1c Calibrator Kit is intended to establish the point of reference in the kinetic determination of serum Hemoglobin A1c in human serum or plasma.

PREPARATION

1. Tear off the plastic seal and carefully remove the rubber cap avoiding the loss of lyophilized material.
2. Using volumetric pipettes, accurately reconstitute the lyophilized material with 0.5 ml purified water.
3. Place the removed cap at step 1 on the vial and let the mixtures stand at the room temperature for 20 minutes.
4. Swirl the bottle gently to ensure homogeneity but avoiding producing bubbles before use.
5. Mix 20 μ l diluted solution with 1ml purified water to make the working solutions.

STORAGE AND STABILITY

Store the unopen kit at 2-8°C till the expiry date. The diluted liquid is stable at 2-8°C for 30 days or at -20°C for up to 6 months. Avoid direct sunlight and repeated freeze-thaw cycle. The working solution should be run out in 30 days by keeping at 2-8°C.

PRECAUTIONS

1. The Kit is for *in vitro* diagnostic use only. Not for use in humans or animals.
2. The instructions must be followed to obtain accurate results.
3. Do not use the reagents beyond the expiration date.
4. In case of accidental ingestion, rinse the mouth and throat with excess water and immediately call for medical attention.
5. For safe waste disposal, it is recommended that each laboratory complies with established laboratory procedures and local federal regulations.
6. The material of the product is human blood with HbsAG, HCV, HIV-1, HIV-2 and Syphilis negative tests. Be sure to use personal protection while using it.

PERFORMANCE CHARACTERISTICS

Please see the values printed on the bottle label.

Not Intended for Sale in the United States.

Pointe Biotech (Nanjing) Co., Ltd.

No.85,Xingmin South Road, Jiang Ning District, Nanjing,P.R.China 211100

Tel:86-25-52425019-800,

Fax:86-25-52425019-866

info@biowaydx.com

