

Bioway Chemistry Calibrator Series Microalbuminuria Calibrator Kit

1*1 ml Liquid



Cat. No. R045S00

MALB calibrator Kit

INTENDED USE

Bioway Chemistry Calibrator Series Microalbuminuria Calibrator Kit is intended to establish the point of reference in the kinetic determination of microalbuminuria in human serum or plasma.

PREPARATION

1. It is a ready-to-use control kit and no need for extra preparation before use.
2. Follow the instructions below to make the series diluted solutions for a multi-point calibration.

Calibrator (μl)	0	10	20	40	80	160
0.9% NaCl (μl)	160	150	140	120	80	0
Dilution factor	0	0.0625	0.125	0.25	0.5	1

STORAGE AND STABILITY

Store the kit at 2-8°C till the expiry date. Avoid direct sunlight.

PRECAUTIONS

1. The Kit is for *in vitro* diagnostic use only. Not for use in humans or animals.
2. The instructions must be followed to obtain accurate results.
3. Do not use the reagents beyond the expiration date.
4. In case of accidental ingestion, rinse the mouth and throat with excess water and immediately call for medical attention.
5. For safe waste disposal, it is recommended that each laboratory complies with established laboratory procedures and local federal regulations.

PERFORMANCE CHARACTERISTICS

Please see the values printed on the bottle label.

Not Intended for Sale in the United States.

Pointe Biotech (Nanjing) Co., Ltd.

No.85,Xingmin South Road, Jiang Ning District, Nanjing,P.R.China 211100

Tel:86-25-52425019-800,

Fax:86-25-52425019-866

info@biowaydx.com

