

波音特生物科技（南京）有限公司

日立 (HITACHI 7170)

Apo-E

| | Analyze | Calib. | Range | STD Conc | | |
|-------------------|-------------|---------------|-----------|----------|-----|-----|
| Assay/Point | 2 Point End | 10 | 16 | 34 | 0 | 0 |
| Wave(Sub/Main) | 700 | 340 | | | | |
| S. Vol. (Normal) | 4.0 | 0.0 | 0 | | | |
| S. Vol. (Dec.) | | 0.0 | 0 | | | |
| S. Vol. (Inc.) | 6.0 | 0.0 | 0 | | | |
| Diluent | Water | 0 | | | | |
| Reagent Vol. (R1) | 210 | 0 | # | 0 | | |
| Reagent Vol. (R2) | 0 | 0 | 00000 | 0 | | |
| Reagent Vol. (R3) | 70 | 0 | # | 0 | | |
| Reagent Vol. (R4) | 0 | 0 | 00000 | 0 | | |
| ABS. Limit | 32000 | Inc. | Twin Test | | | |
| Prozone limit | 0 | 0 | Lower | | | |
| Cell Det. | Detergent 1 | | | | | |
| | Analyze | Calib. | Range | STD Conc | | |
| Calib. Type | SPLINE | | | | | |
| Point | 6 | Span Point | | 3 | | |
| Weight | 0 | | | | | |
| Auto Calibration | | | | | | |
| | Timeout | | Chg. Over | | | |
| Blank | 0 | Change Lot | | | | |
| Span | 0 | Change Bottle | | | | |
| 2Point | 0 | | | | | |
| Full | 0 | | | | | |
| SD Limit | 999 | | | | | |
| Duplicate Limit | 500 | | | | | |
| Sensitiv. Limit | 0 | | | | | |
| S1ABS Limit | -32000 | 32000 | | | | |
| | Analyze | Calib. | Range | STD Conc | | |
| <Standard> | (1) | (2) | (3) | (4) | (5) | (6) |
| Conc. | 0.0 | # | # | # | # | # |
| Position Water | # | # | # | # | # | # |
| Volume | 4.0 | 4.0 | 4.0 | 4.0 | 4.0 | 4.0 |
| <Pre-Diluent> | | | | | | |
| Volume | 0.0 | 0.0 | 0.0 | 0.0 | 0.0 | 0.0 |
| Diluent | 0 | 0 | 0 | 0 | 0 | 0 |
| Calib. Code | 0 | 0 | 0 | 0 | 0 | 0 |

“#” 由使用者自行输入浓度或位置。

地址：南京市江宁区科学园兴民南路85号

电话：025-52425019

传真：025-52424836-807