

# 波音特生物科技（南京）有限公司

日立 (HITACHI 7170)

CK-MB

		Analyze	Calib.	Range	STD Conc	
Assay/Point	RateA	10	23	31	0	0
Wave(Sub/Main)	405	340				
S. Vol. (Normal)	10.0	0.0	0			
S. Vol. (Dec.)		0.0	0			
S. Vol. (Inc.)	6.0	0.0	0			
Diluent	Water	0				
Reagent Vol. (R1)	200	0	#	0		
Reagent Vol. (R2)	0	0	00000	0		
Reagent Vol. (R3)	50	0	#	0		
Reagent Vol. (R4)	0	0	00000	0		
ABS. Limit	32000	Inc.	Twin Test			
Prozone limit	0	0	Lower			
Cell Det.	Detergent 1					
		Analyze	Calib.	Range	STD Conc	
Calib. Type	LINEAR					
Point	1	Span Point		0		
Weight	0					
Auto Calibration						
	Timeout			Chg. Over		
Blank	0	Change Lot				
Span	0	Change Bottle				
2Point	0					
Full	0					
SD Limit	999					
Duplicate Limit	500					
Sensitiv. Limit	0					
S1ABS Limit	-32000		32000			
		Analyze	Calib.	Range	STD Conc	
<Standard>	(1)	(2)	(3)	(4)	(5)	(6)
Conc.	0	0	0	0	0	0
Position	Water	0	0	0	0	0
Volume	10.0	0.0	0.0	0.0	0.0	0.0
<Pre-Diluent>						
Volume	0.0	0.0	0.0	0.0	0.0	0.0
Diluent	0	0	0	0	0	0
Calib. Code	0	0	0	0	0	0

“#”由使用者自行输入浓度或位置。理论因子 F 为 8360，需根据仪器本身用定值血清修正。

地址：南京市江宁区科学园兴民南路85号

电话：025-52425019

传真：025-52424836-807