

波音特生物科技（南京）有限公司

日立 (HITACHI 7170)

LDL-C

	Analyze	Calib.	Range	STD Conc		
Assay/Point	2Point End	10	16	34	0	0
Wave(Sub/Main)	660	546				
S. Vol. (Normal)	2.0	0.0	0			
S. Vol. (Dec.)	3.0	0.0	0			
S. Vol. (Inc.)	6.0	0.0	0			
Diluent	Water	0				
Reagent Vol. (R1)	210	0	#	0		
Reagent Vol. (R2)	0	0	00000	0		
Reagent Vol. (R3)	70	0	#	0		
Reagent Vol. (R4)	0	0	00000	0		
ABS. Limit	32000	Inc.	Twin Test			
Prozone limit	0	0	Lower			
Cell Det.	Detergent 1					
	Analyze	Calib.	Range	STD Conc		
Calib. Type	LINEAR					
Point	2	Span Point		2		
Weight	0					
Auto Calibration						
	Timeout		Chg. Over			
Blank	0	Change Lot				
Span	0	Change Bottle				
2Point	0					
Full	0					
SD Limit	999					
Duplicate Limit	500					
Sensitiv. Limit	0					
S1ABS Limit	-32000	32000				
	Analyze	Calib.	Range	STD Conc		
<Standard>	(1)	(2)	(3)	(4)	(5)	(6)
Conc.	0.00	#	0	0	0	0
Position	Water	#	0	0	0	0
Volume	2.0	2.0	0.0	0.0	0.0	0.0
<Pre-Diluent>						
Volume	0.0	0.0	0.0	0.0	0.0	0.0
Diluent	0	0	0	0	0	0
Calib. Code	0	0	0	0	0	0

“#” 由使用者自行输入浓度或位置。

南京市江宁区科学园兴民南路85号

电话：025-5242501

传真：025-52424836-807